

Promoting Best Practice

An example of a BionectorTKO® dressing change pack:

Component	Quantity	Benefit
Outer wrap / trolley drape	1	Provides a sterile field for products
BionectorTKO®	1	Helps protect IV catheter from infection and occlusions
Gloves (non-sterile)	1 pair	For dressing removal
Mask	1	Maximum barrier precautions pre-dressing removal
Apron	1	Maximum barrier precautions
Alcohol cleaning pad(s)	3+	To remove the dressing, securement device and for BionectorTKO® disinfection
Tape strips	1 pack	To secure the catheter during cleaning and post cleaning
Gloves (sterile)	1 pack	For dressing replacement
Under arm drape	1	Sterile field for the insertion site
ChloraPrep® 3ml applicator	1	Skin disinfection
BIOPATCH®	1	To prevent extra-luminal contamination
Securement device (Grip-Lok™)	1	To secure the catheter
CHG dressing	1	To protect the insertion site
Luer-lock syringe (10ml)	2	For patency check (withdrawal and flush) and final lock flush
10ml pre-filled syringe with NaCl 0.9%	1	For flushing the IV line
Blunt fill drawing-up needle(s)	2	For drawing up the flush for patency check and for giving final lock
120ml cristal round gallipot	1	For dispensing solutions
Swabs 10 x 10cm, 4 ply, non-woven	5	General purpose
Yellow disposal bag 28x45cm	1	For equipment disposal
Documentation sticker	1	To document the process

References

1. <http://webarchive.nationalarchives.gov.uk/20120118164404/http://hcai.dh.gov.uk/files/2011/03/2011-03-14-HII-Peripheral-intravenous-cannula-bundle-FIN...pdf> - accessed on 15/01/2016
2. County Durham and Darlington NHS Foundation Trust Bionector TKO® Study - information available on request
3. H. P. Loveday et al; 2014; The Journal of Hospital Infection; vol. 86, supplement 1, page S11: 'epic3: National Evidence-based Guidelines for Preventing Healthcare-Associated Infections in NHS Hospitals in England'.



This literature has been made using carbon balanced paper. If you would like to find out more about what Vygon (UK) Ltd does for the environment, please visit: www.vygon.co.uk/company/corporate

For further information, please contact: vygon@vygon.co.uk

The specifications shown in this leaflet are for information only and are not, under any circumstances, of a contractual nature. This document is intended for use in the UK only.

Vygon (UK) Ltd • The Pierre Simonet Building • V Park • Gateway North
 • Latham Road • Swindon • Wiltshire • SN25 4DL
 Tel: 01793 748800 • Fax: 01793 748899 • Twitter: @vygonuk
 Web: www.vygon.co.uk Content correct as of: 01/2016 Code: DXJB0043002
 Copyright Vygon (UK) Ltd 2016

ChloraPrep® is a registered Trade Mark of CareFusion Corporation
 Full prescribing information for ChloraPrep® can be found at www.carefusion.co.uk
 SorbaView® is a registered Trade Mark of Centurion Medical Products Corporation.



www.vygon.co.uk



Value Life



**Needle-free VYSET®
 Procedure Packs**
 Promoting best practice

www.vygon.co.uk

vygon@vygon.co.uk

Needle-free VYSET® Procedure Packs

VYSET®

Clinicians face a number of challenges when selecting equipment for an IV procedure:

- **Time wasted** searching for the correct products
- **Return visits** to collect additional items.



With a ready-made procedure pack you have **all** the components needed.



Having all the components in one pack **saves crucial time** for nurses.

We have a solution to this common problem:

The VYSET procedure pack includes all the equipment needed in one handy pack.

A VYSET procedure pack:

- ✓ Promotes standardised procedures
- ✓ Reduces infections
- ✓ Saves time
- ✓ Saves money.

A Range of Options

Our VYSET needle-free procedure packs include:

- Cannulation packs:
 - Single and double lumen
 - Neonatal
 - Paediatric
- PICC and Midline dressing change packs
- Blood culture packs.

Meet National Guidelines

Our packs meet national guidelines that ensure maximum barrier precautions...

...this includes **High Impact Intervention guidelines¹** and research:



Bionector TKO® is our occlusion-prevention device and is **supported by independent research.²**



A range of **chlorhexidine gluconate dressings** are available, including BIOPATCH®.



Pre-filled saline syringes are available for flushes.

Our packs also support **epic3 guidelines³** recommended for asepsis and dressing:



The MHRA recommends the use of **non-return valves**.



Current guidelines recommend the use of chlorhexidine with 70% alcohol to **ensure the patient's skin is disinfected**.



Our range of drapes help prevent **microbial contamination**.

“ Healthcare-associated infections (HCAs) have become a significant threat to patient safety. Many, although not all, can be prevented. **Clinical effectiveness** (i.e. using **prevention measures** that are based on reliable evidence of efficacy) is a core component of an effective strategy designed to **protect patients from the risk of infection**, and when combined with quality improvement methods can account for **significant reductions in HCAs.³** ”



Choose from a variety of sterile, semi-permeable, transparent **dressings**.



Option to have **customised documentation**.